


## Verrutop

Verrutop is a topical solution for professional use providing painless treatment of difficult warts that is highly effective in just a few applications.

**With Nitrizinc Complex:**

A unique, elevated combination

- Nitric acid
- Organic acids
- Zn and Cu salts

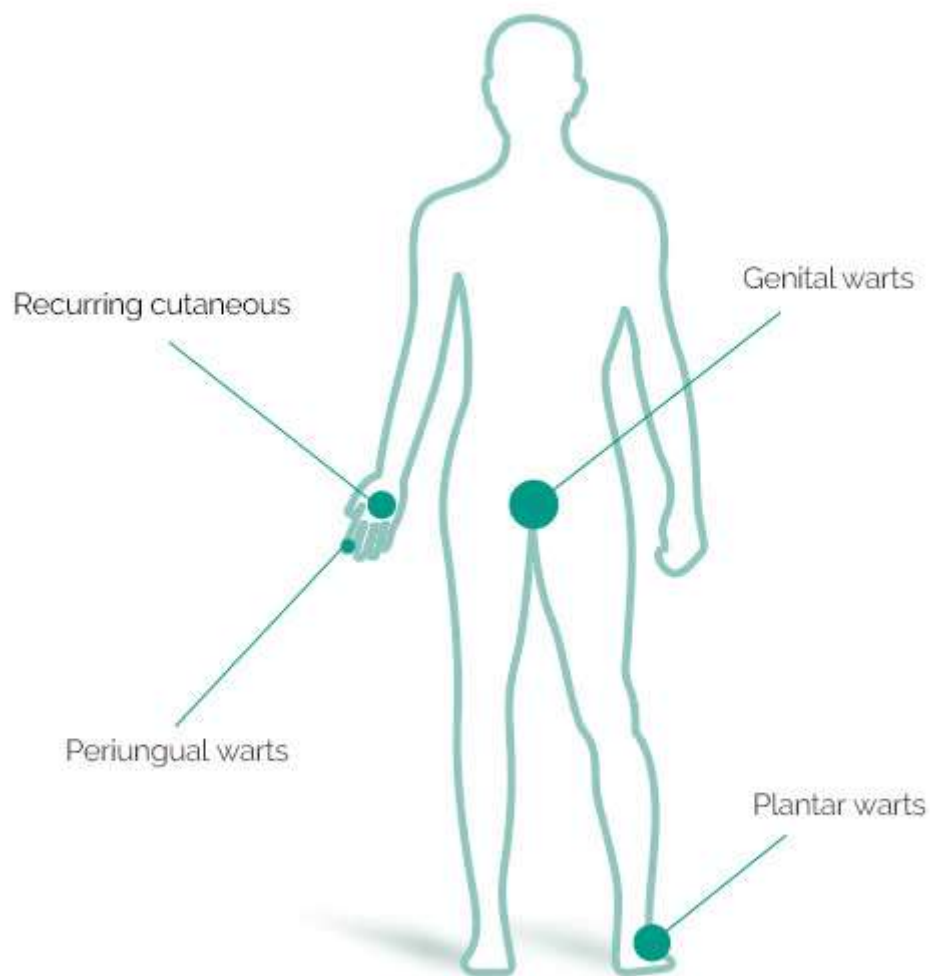

\*As sensitivity varies, some patients may experience a slight, short-lived tingling after application of the product. This disappears after a few seconds.


Use under medical supervision

Class IIA Healthcare Product

Instruction booklet

How do you apply Verrutop?


Verrutop is easy and quick to apply and doesn't affect the patient's daily life.

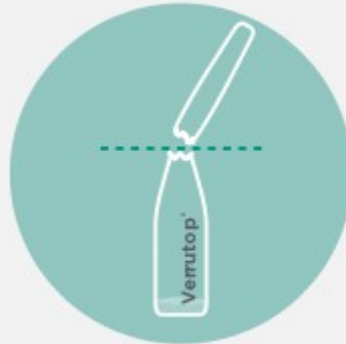

Break the ampoule



Insert the capillary  
to extract Verrutop®

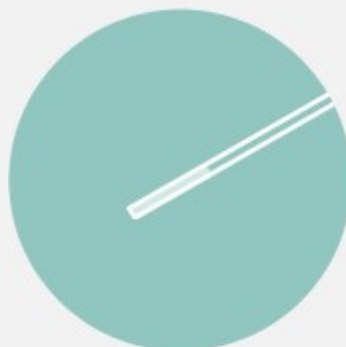

Fill the applicator  
with the product




Apply the product  
with the capillary directly on the wart

Apply the necessary quantity of product to cover the wart according to its size but always avoid excess. If re-application is necessary wait 15 to 30 seconds between applications. The patient must be assessed by a health professional after a minimum period of 1 to 2 weeks, depending on the location to evaluate elimination of the wart or to re-apply the product if necessary. Treatment could be repeated for up to a maximum of six sessions, separated by a minimum period of 7 days. Check the [product information leaflet](#) for further information and warnings.

See how to eliminate verrucas with Verrutop® with Dr Ogalla. [See video](#)

Quick and painless\* application

One applied to the warts, it produces a painless mummification of the damaged tissue that causes the wart to detach automatically as healthy skin forms.

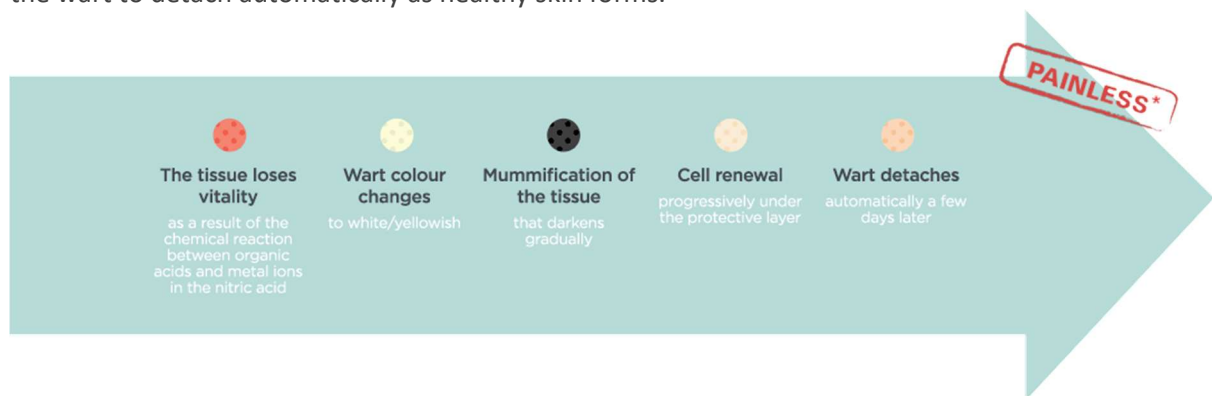

Proven efficacy,  
even in the most difficult cases



90%

of patients cured after a maximum of 6 treatment sessions<sup>1</sup>. Average: 3 sessions .

87%

of patients cured after a maximum of 4 treatment sessions<sup>2</sup>. 49% of them cured after only one session

Efficacy in palmoplantar warts resistant to cryotherapy<sup>3</sup>

72%


responded totally or partially to treatment with Verrutop<sup>®</sup>




% cured plantar warts

59%

disappeared after 1-3 applications of Verrutop<sup>®</sup>

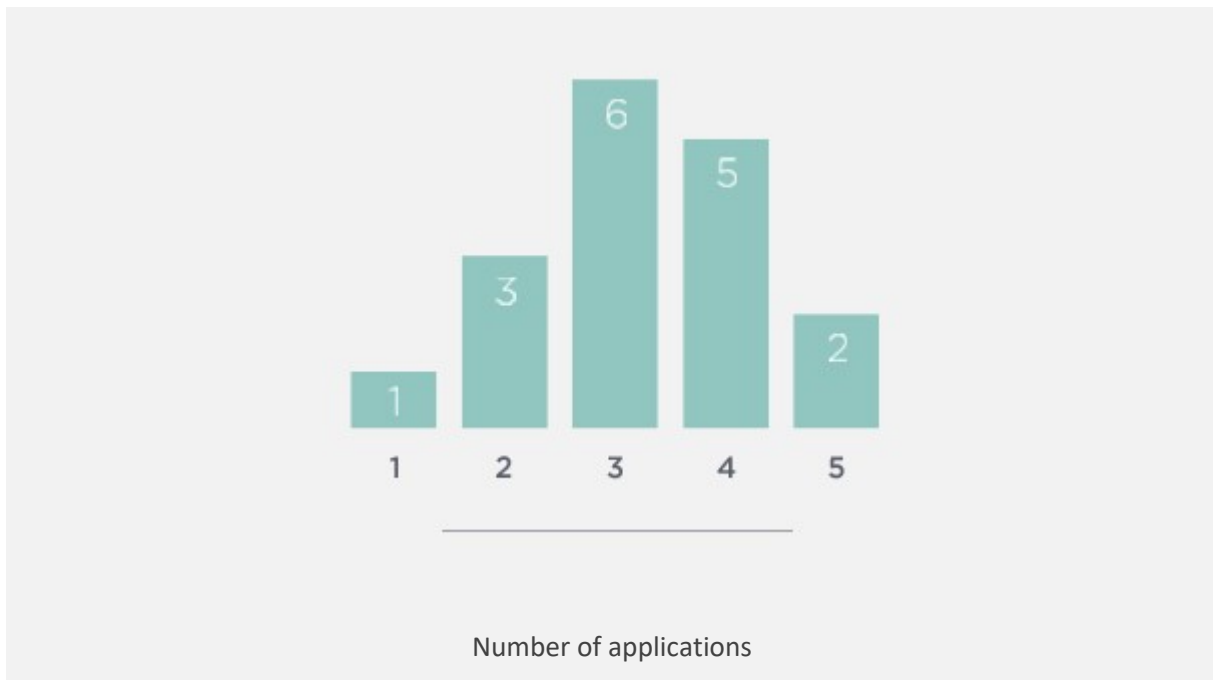

With Verrutop, the complete viral DNA elimination occurs in 40% of cases, while no effect was achieved by liquid nitrogen<sup>4</sup>

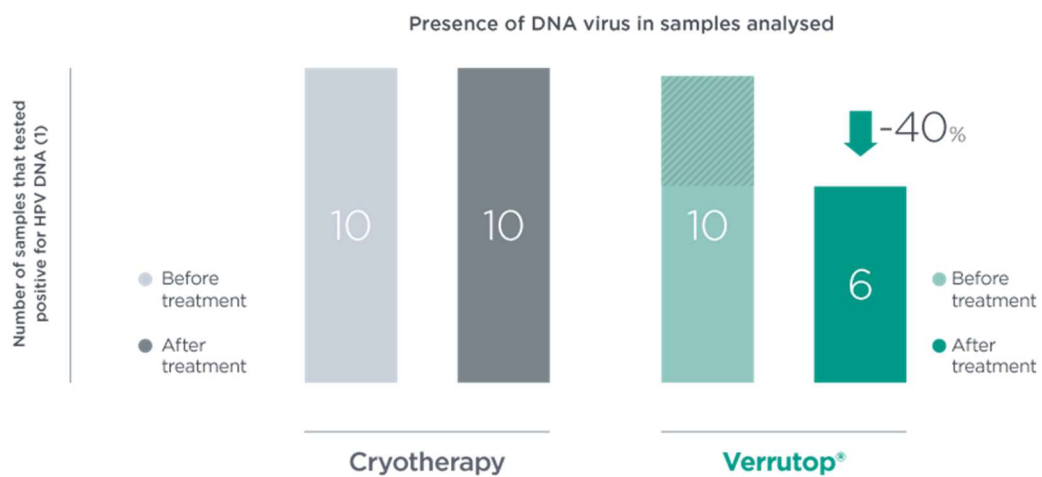

#### References

- \*. Verrutop in general is a painless treatment, although some people may experience some temporary discomfort that ranges from a burning to slight tenderness or pain.
- Galloni C, Balice Y, Fachetti M, Marchetti S, Motta L, Pisacane D. Efficacy and tolerability assessment of a new medical device containing a nitric-zinc complex to treat palmar, plantar and periungual warts: experience with 210 patients. Poster session presented at 27th EADV Congress; 2018 Sep 12-16; France, Paris.
  - Ciccarese G. Efficacy Assessment of a Topically Applied Nitric-Zinc Complex Solution for the Treatment of External Anogenital Warts in 100 Patients. *Dermatol Ther (Heidelb)*. 2019 Jun;9(2):327-335.
  - Guillet G, Jéroche F. Evaluation de l'efficacité et de la tolérance d'un traitement de verrues palmo-plantaires après échec thérapeutique avec cryothérapie, et de verrues péri-unguérales. Poitiers, France, *Dermatologie pratique* N°363-octobre 2012, p.9-10.
  - Viennet C et al. Assessment of the efficacy of a new formulation for plantar wart mummification: new experimental design and human papillomavirus identification. *CED* a 2012 British Association of Dermatologists *Clinical and Experimental Dermatology*, 38, 85-88